

Tracer *Mycoplasma bovis*

Reference: BDE K 302

Internal reference material for *Mycoplasma bovis* serology by ELISA

For veterinary *in vitro* use only



Pathology concerned

Bovine mycoplasmosis.

Designation

Internal reference material for bovine mycoplasmosis serology by ELISA.

Product reference

BDE K 302 - 1 vial of 0,5 mL

Method of production

Material obtained from a *Mycoplasma bovis*-seropositive cattle.

Intended use

This material can be used to evaluate and verify the fidelity of analyses. It can be used, for example, as a tracer for control charts.

Refer to the certificate of analysis for target values.

Instructions for use

It should be treated as a sample. Dilutions are to be made with the dilution solution of the kit in use. 1/100 dilution for BIO K 302 and for BIO K 260.

Storage

Keep refrigerated at 5°C (± 3° C).

Special biosafety measures

No specific measures (see MSDS).

Disposal

In accordance with user laboratory procedures.

Revision history

Date	Version	Modifications
02/07/2026	V01	First version

Note : minor typographical, grammar and formatting changes are not included in the revision history.