



# OPTIMAL CONDITIONS OF USE

## Validation of Analysis

### Valid analysis

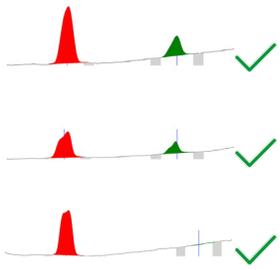


### Invalid analysis

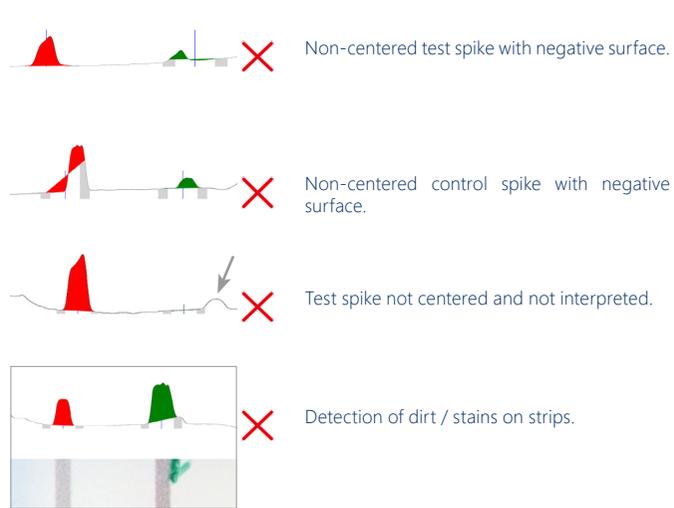


## Drop shadow



## Hiding the QR Code & Reading Area

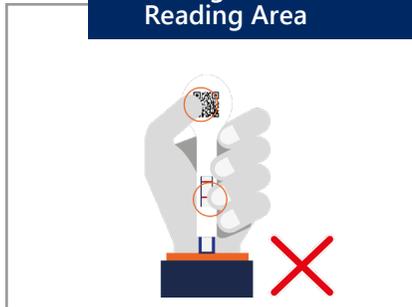

## Parallax of Smartphone and SmartStrips

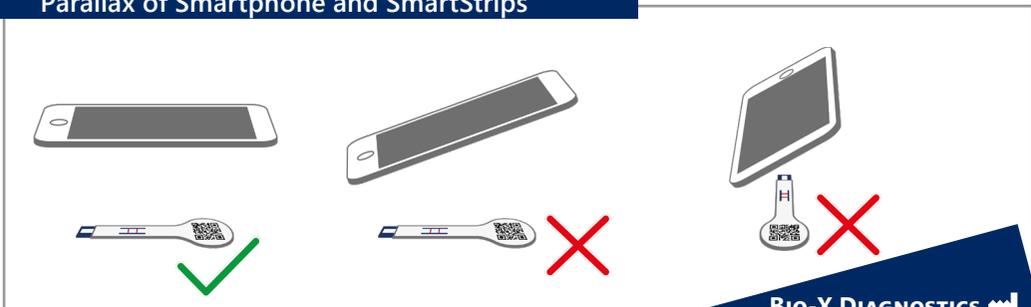

### Bio-X DIAGNOSTICS

38, rue de la Calestienne  
5580 Rochefort • BELGIUM  
T. +32(0)84 32 23 77 • F. +32(0)84 31 52 63  
info@biox.com • www.biox.com

EN



User's manual - iOS 

Date : 13/09/2019  
Version : 3.1

**SMARTSTRIPS**  App




Smart solutions for shar decisions



Bio-X Diagnostics is ISO 9001:2008 certified to assure the best to its customers



## Adding a test

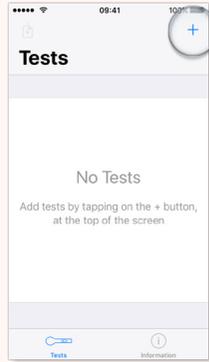




■ Launch SmartStrips™ App on your smartphone.

■ Add tests by touching the "+" button at the top of the screen.

■ The application detects the SmartStrips™ device automatically and captures the test area.

## Creating/Exporting a results file

### 1 Creating a results file

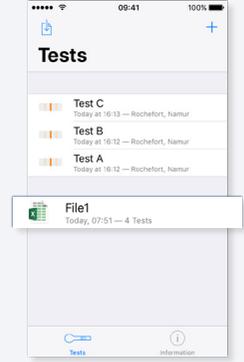
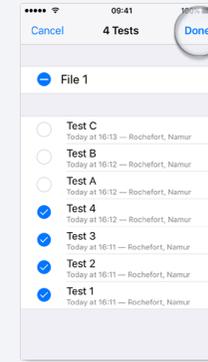
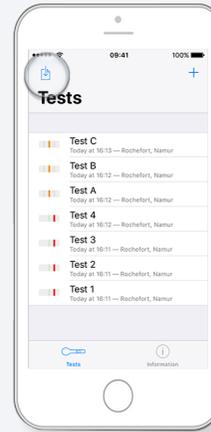


■ Start by creating a file.

■ Select all of the tests icon include in the file that you want to create and rename the file.

■ Your file has been created.

## Reading the results

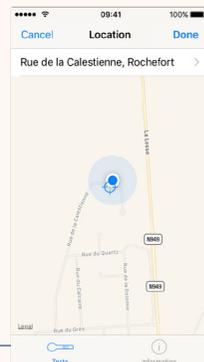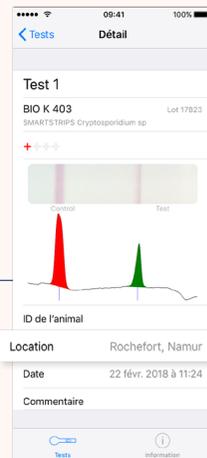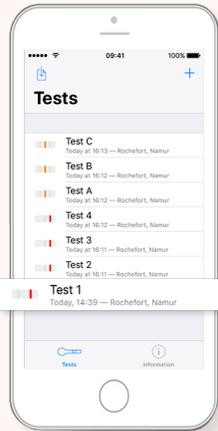


■ Open a test.

■ The Detail page gives all the general test information.

■ The Location page gives the location of the test.

### 2 Exporting a results file

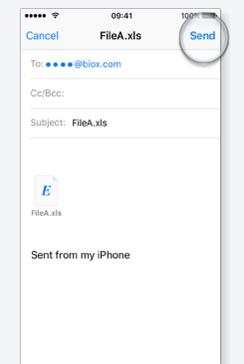

■ Open the file.

Touch the export icon in the upper right-hand corner of the screen.

■ Touch the icon for sending e-mail.

■ Enter the recipient's e-mail address and type your message.

Touch "Send" to send your e-mail.

Consult your results on a computer.